SuperEar® SE5000

SonicTechnology®
Quality and Innovation since 1981

The SuperEar SE5000 has surface mount microelectronics for crystal clear amplification of ambient sounds by up to 50 decibels, for a total decibel gain of 107 dB.

SuperEar® Model SE5000 utilizes (1) AAA Alkaline Battery (Included).

The SuperEar® performs at optimum with **Energizer** or **Duracell** batteries. If static occurs during use, this is typically symptom of battery performance. Changing the battery solves most static issues.

The battery door is located in the front of the unit. Place the bottom of the battery on the spring located in the battery compartment. The left hand side battery has the spring on the bottom/the right hand side has the spring on the top. Please see diagram below.

The Volume Control Wheel is located on the side of the unit. Please always confirm the unit is in the OFF position before placing the headphones on the head. Once the headphone is comfortably positioned, slowly advance the Volume Control to the desired amplification.

Please refer to our cleaning instructions attached for cleaning information and for Disposable Sanitary earpad covers.

Place the Bottom of the Battery on **spring**

Confirm the **OFF** position on the Volume Control Wheel before attaching the headphones to the amplifier.

Headphone Jack

Disposable Sanitary Earpad Cover

Our SuperEar® team is available to help. Contact us:
superear@sonictechnology.com
or 800-247-5548 8am-5pm PST Mon- Thurs.

Manufacturer Suggested Cleaning Recommendations

CMS 3.0 and ADA compliant Personal Sound Amplifiers
SuperEar® Models SE5000, SE7500, SE9000HP

The SuperEar® amplifier and accessories can be cleaned with a light application of an alcohol-based disinfectant.

Caustic cleaning agents can prevent health risks but reduce headphone life. All SuperEar® hearing amplifiers come packaged with a pair of comfort foam ear pads on the included headphones to help keep individuals comfortable during extended periods of use. Replacement headphones (SE-HP) and foam pads (EPF200) are available for purchase, however, should something happen while the product is under warranty, please contact us.

In a hospital or long-term care environment, disposable headphone earpad covers (EPC100) can provide economical and inexpensive shields from cross-contamination for multiple headphone users while still providing sound clarity. EPC100 are sanitary disposable headphone earpad covers for small headphones. Soft, comfortable, inert fabric allows MRI scans, and is hypoallergenic, perfect for hospital patients. Our hygienic breathable material allows clear sound quality while maintaining sanitary inter-person use. Easy to apply and replace after each use, Ear Pad Covers can be removed and discarded. The plastic portion of the headphones can be cleaned with a light application of an alcohol based disinfectant.

For added convenience and control, Sonic Technology disposable covers come in a box of 100 covers (50 pairs) for easy storage and distribution.

EPC100 Disposable Hygienic Earpad Covers

- EPC100 are Ideal for hospital, home, and healthcare environments
- Elastic stretches and secures on most small headphones
- Fits personal headphones of SE5000, SE7500, and SE9000HP
- Earpad covers are conveniently packaged 100 individual covers (50 pair) per box
- Hypoallergenic woven material is breathable and stretches 2.5" inches in diameter

SonicTechnology
Quality and Innovation since 1981

It is always best to consult with your Infection
Control Officer regarding proper cleaning procedures

www.sonictechnology.com
superear@sonictechnology.com

FAX 530-272-4257 | 1-800-247-5548
P.O. Box 539, Grass Valley, CA 95945

SuperEar®

Model SE5000

Personal Sound Amplifier

Owner's Manual

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Manuale del proprietario

Instructions for Use

1. To remove the battery compartment cover, gently depress the cover with your thumb and slide it in the direction of the arrow.
2. Insert the AAA 1.5v battery. Match the positive (+) and negative (-) ends to those indicated on the battery cavity. Alkaline batteries will deliver 2-3 times the operational life of lesser batteries.
3. Slide the battery compartment cover back into locked position.
4. Insert the earphone connection into the jack.
5. Be sure the amplifier switch is in the OFF position before putting on the earphones. Adjust headband to fit comfortably.
6. Slowly turn the volume control wheel to the desired level of amplification.
7. Attach the pocket clip to the bottom of the amplifier using the screw provided. Do not over-tighten. Can be used on pocket, belt, or other gear.

User Information

1. This amplifier has a maximum gain of 50dB. At approximately ¾ open on the volume control, the device is at full volume. If you turn the volume control all the way up, it may result in an annoying feedback noise. Simply turn the volume control down to eliminate feedback.
2. If headphones are placed near the amplifier while it is turned on, feedback squeal will result. Always turn your SuperEar® off before you remove your headphones.
3. The SuperEar® requires only a single 1.5AAA alkaline battery for up to 30 hours of operation. Battery included.
4. Premium stereo headphones and earbuds deliver clear, crisp sound particularly well in the human voice range. The foam ear pads are easily replaceable.

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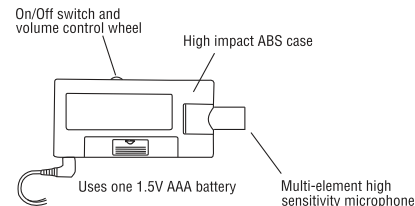
3 YEAR WARRANTY

Sonic Technology Products, Inc. warrants to the original purchaser that your SuperEar® will be free of defects in material or workmanship for a period of 3 years from the date of purchase. Warranty is void if defects are caused by negligence, abuse, misuse or damage caused by water or heat.

This warranty gives you specific legal rights and you may have other rights which may vary from state to state.

Please call us first to determine if it will be necessary to send your SuperEar® in for service under the terms of this warranty. If yes, it must be returned in a well-packaged carton and shipped postage prepaid.

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530-272-4607
www.sonictechnology.com
www.superear.com

Rev. 12/19

Setup in 4 Simple Steps for Instant Use

- 1** *Insert included AAA battery to power on.*

- 2** *Connect earbuds or headphones for clear sound.*

- 3** *Adjust volume using the on/off wheel.*

- 4** *Attach the belt clip for hands-free convenience.*

SuperEar®

SE5000

Personal Sound Amplifier

HANDHELD SOUND AMPLIFIER

Automatic Shut Off Microphone	No
Body frame material	High Impact ABS
Max Output	107 dB
Humidity Operating Range	At 20°C, 95% RH
Operating Temperature	0°C - 55°C
Frequency Range	100Hz - 14kHz
Plug-in connector	Stereo Jack, 3.5mm diameter
Power Source	One AAA Alkaline battery (included)
Weight	3 oz.
Belt/Pocket Clip	Included

HEADPHONES (SE-HP)

Diaphragm	Mylar, diameter 30mm.
Impedance	32 ohms +- 20% at 1kHz
Nominal Power	20mW
Max Power	40mW
Sensitivity	100 dB S.P.L +- 2dB 1kHz
Balance per channel	+/- 3 dB at 1kHz
Frequency Response	20Hz-20kHz
Plug-in connector	Stereo Plug, 3.5mm diameter
Cord Length	1150mm - +-50mm
Weight	50g +-5g
Earpad	Black, cushioned sponge

EARBUDS (LC-EB)

Diaphragm	Mylar, diameter 10mm.
Impedance	32 ohms +- 15% at 1kHz
Nominal Power	5mW
Max Power	10mW
Sensitivity	105 dB S.P.L +- 3dB 1kHz
Balance per channel	+/- 3 dB at 1kHz
Frequency Response	20Hz-20kHz 1kHz
Plug-in connector	Stereo Plug, 3.5mm diameter
Cord Length	1150mm - +-50mm
THD	Less than 5% at 1kHz 0.126V

HS TARIFF CODE: 8515.50

SE5000 DETAILS

WARRANTY	3 year
MEASUREMENTS	3.05"x 2"x 0.75"
CERTIFIED	CE & ROHS
COMPLIANT	CMS, MDS, ADA, ACA 1557, Prop 65

Cleaning instructions available

SonicTechnology®

Quality and Innovation since 1981

Use with Disposable Sanitary Earpad Covers
- Model EPC100

www.sonictechnology.com
superear@sonictechnology.com

FAX 530-272-4257 | 1-800-247-5548
P.O. Box 539, Grass Valley, CA 95945

V1.1 11/20/19